


# SurgiTel Face Shield Usage Instructions


(Patent-Pending Design)

- Push out pre-scored vertical slot if you wish to have your light protrude from shield. If you wish to try your light behind shield, DO NOT punch out vertical slot.
- Face shield sent with protective film to protect from scratching. Remove from both sides before using.
- Usage Duration: Per CDC recommendations, it is okay to use products which are in limited supply as long as they are safe to use. When the shield clarity is reduced, it must be replaced. Customers are to always follow the current CDC recommendations. Customers should check [www.cdc.gov](http://www.cdc.gov) regularly for updates.
- Disinfection: Customers may use any materials they feel is required to meet their disinfection requirements. Optim 1 is recommended.
- If you require additional space behind your face shield for larger loupes, or to use your light without punching out vertical slot: see instructions at the bottom of this page
- Attaching a Universal Clip Adaptor: See instructions on reverse side

See images below for examples of different loupe and light usage.

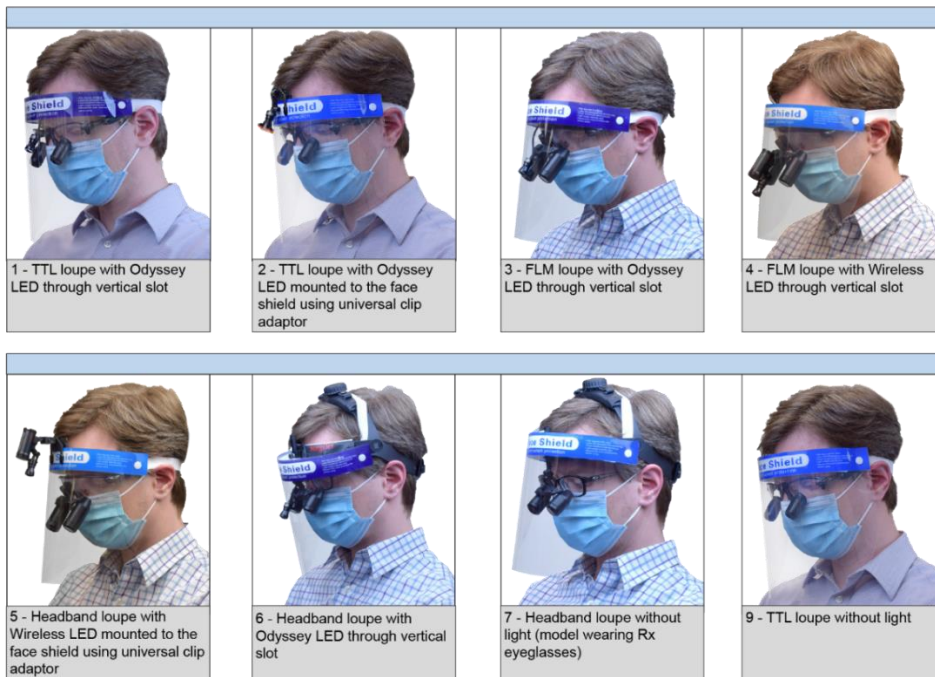

## If you require additional space behind your face shield:

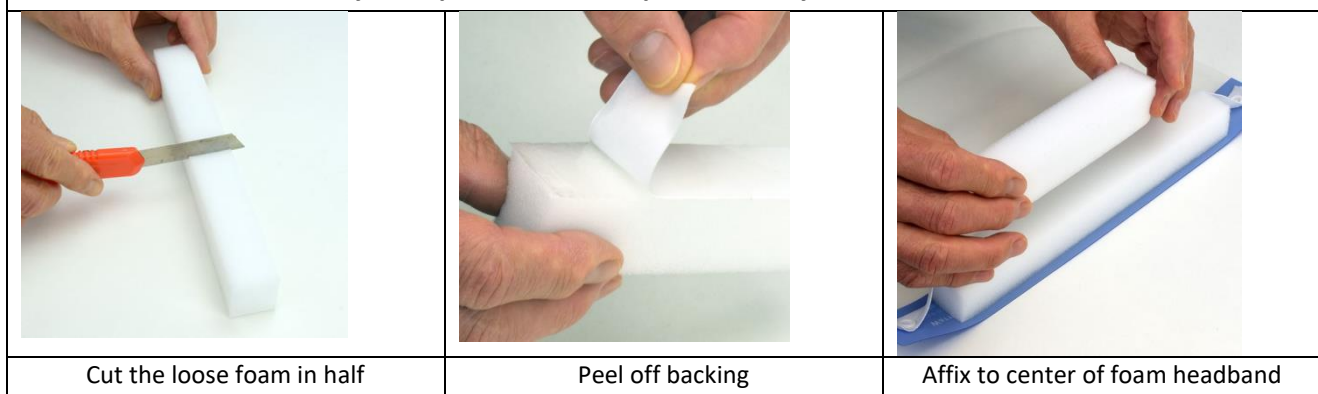







Contact our support team at [Support@SurgiTel.com](mailto:Support@SurgiTel.com) if you have any further questions.

**Thank you for your purchase!**
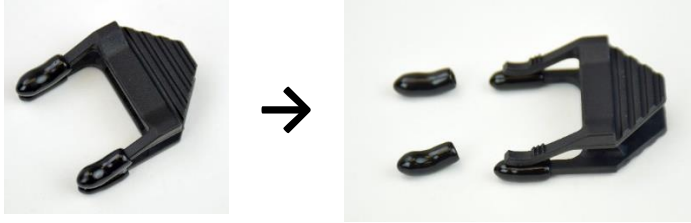
## Instructions for attaching a Universal Clip Adaptor (Optional)

If you are using a Universal Clip Adaptor you will need: your adaptor, your face shield, a pen or marker, and a thin screwdriver or similar instrument.

	
<p>Step 1. If you have a Universal Clip Adaptor with the rubber 'caps,' you may remove the caps off of the inside prongs.</p>	<p>Step 2. Line up your adaptor with the center of your face shield, as indicated by the light hole. (If using adaptor, you do not need to punch out)</p>
	
<p>Step 3. Use a marker or pen to mark where the prongs of the adaptor lines up.</p>	<p>Step 4. Use a thin screwdriver or similar tool to punch a hole at the marking, close to the shield. As always, be careful with sharp objects.</p>
	
<p>Step 5. Clip the adaptor to your face shield as shown.</p>	<p>Step 6. Attach your light to your adaptor. You may now detach/attach the adaptor to your face shield as needed.</p> <p>Optional: If you have the "Coaxial Light Unit Adapter," attach as shown</p>